


# RELENZA®

## ZANAMIVIR

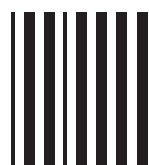


FOR ORAL INHALATION  
IMPORTANT PLEASE READ BEFORE USING

Patient's instructions for use

### STEP BY STEP INSTRUCTIONS FOR USING THE DISKHALER®

#### THE PARTS OF YOUR DISKHALER

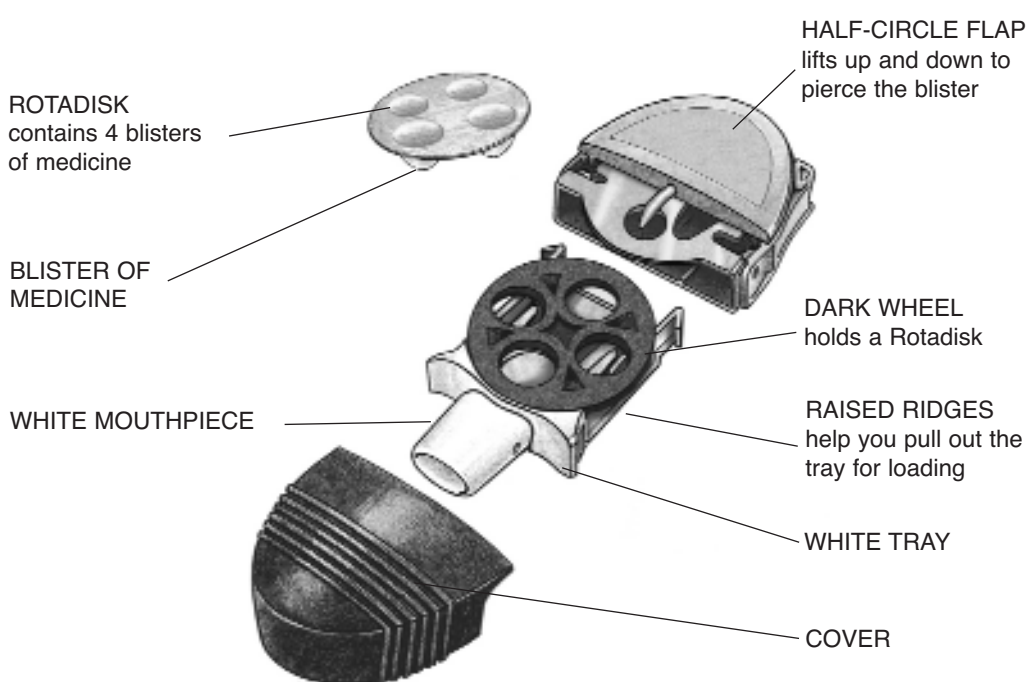

®Relenza, Diskhaler and Rotadisks are registered trade marks of the GlaxoSmithKline group of companies.


#### EASY TO USE PATIENT INSTRUCTIONS

### STEP ONE: LOAD


#### ROTADISK




The first step is to load a Rotadisk® into the Diskhaler®.

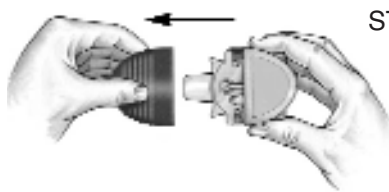


**START BY PULLING OFF THE COVER.**

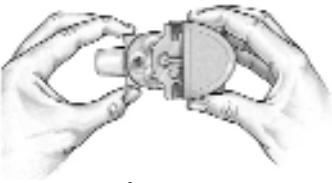


**PULL OUT TRAY**  
Pull out the white mouthpiece by the edges until it stops.

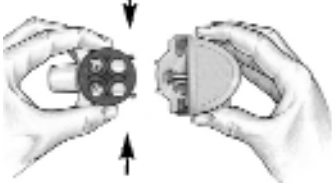

**SQUEEZE AND REMOVE TRAY**  
Find the 4 raised ridges on each side of the white tray. Press in these ridges, and pull the white tray out of the Diskhaler.

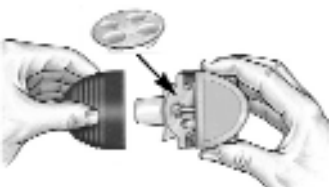

**DROP IN ROTADISK**  
Place one Rotadisk onto the dark wheel.

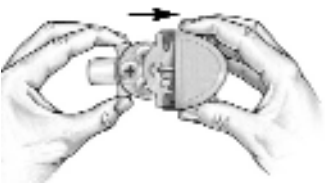

**CLOSE**  
Push in the white tray, now loaded with a Rotadisk, so it clicks firmly back into place.


DO NOT PROCEED TO STEP 2 UNTIL YOU'RE READY TO TAKE YOUR MEDICINE.

### STEP TWO: PUNCTURE

The Diskhaler is designed to puncture one small blister of medicine at a time. It doesn't matter which blister you start with.

**BE SURE TO KEEP THE DISKHALER LEVEL**

#### LIFT AND PUNCTURE

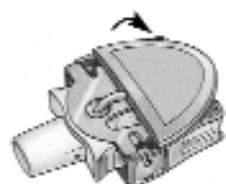


Locate the half-circle flap on top of the Diskhaler. Lift this hinged flap to the fully upright position to pierce the blister. The flap must be straight up to properly pierce the silver foil of the Rotadisk. Push the flap back down into place.

### STEP THREE: INHALE



Put the white mouthpiece into your mouth being sure not to cover the small holes on either side of the mouthpiece. Close your lips firmly around it. Take a quick, deep breath through the mouthpiece to breathe in the medicine.

### STEP FOUR: NEXT DOSE

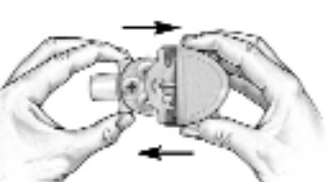

For the next dose of medicine, just pull the mouthpiece to extend for white tray, but don't remove it. Then push it back until it clicks. The PULLOUT/PUSH IN motion makes the Rotadisk rotate to the next blister. Repeat steps 2 and 3 to take a second dose. If all 4 blisters in the Rotadisk have been punctured, the Diskhaler is ready to be reloaded with a fresh Rotadisk.